



# The Paww

User Guide



[www.yourkoalaa.com](http://www.yourkoalaa.com)

# Let Your Koalaa Journey Begin...

It's so great to meet you and we at Koalaa can't wait for you to use your Paww. This booklet has all you need to know to get started but get in touch with your limb buddy any time if you have any questions on **[hi@yourkoalaa.com](mailto:hi@yourkoalaa.com)**





# Say Hello To Your Paww!

This dynamic soft prosthetic is made up of a flexible sleeve with a soft fabric cover to hold it in place. The Paww is suitable for a **wide range of limb differences**, including many that may have been turned away from more traditional prosthetics.


The Paww is also compatible with a wide range of detachable tools offering access to countless activities.

## What's included in your package?



### The Paww

The backbone of your Koalaa Paww is the blank.

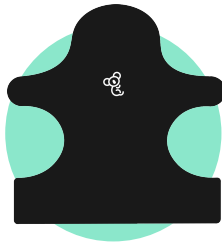

### Paww Cover








The Paww is secured in place with a soft, fabric cover.



### Paww Tools

Your tool/s of choice to clip into the Paww and help with your favourite activities.

# How to wear **The Paww**

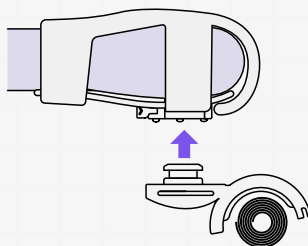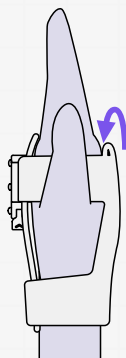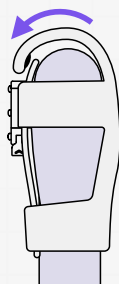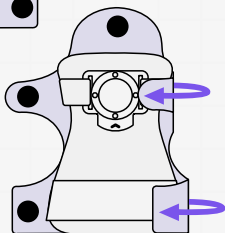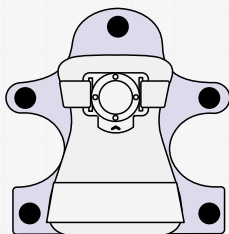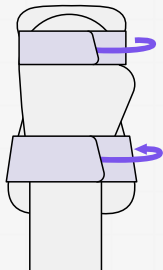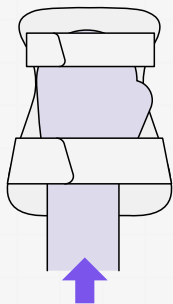

> **10kg**



# Taking Care Of Your Paww

## Safety



### Comfort

Ensure the sleeve is comfortable and not too tight on your limb. If the Paww causes marking on your skin that does not go away after 20 minutes then the sleeve is too tight and should be loosened.



### Dock Position

Ensure the Paww is in the correct position with the tool release button sitting under your palm or limb.



### Take Care

Keep all digits clear of the dock when attaching tools to the Paww.

## Maintenance And Cleaning



### Cleaning

Regularly clean your tools by wiping any excess dirt with a damp cloth.



### Machine Wash

Wash your Paww blank and cover by removing the tool and then popping it in with your regular wash at 30 degrees.

Top tip! Put your Paww into a bag or pillowcase to protect the other items in your machine.







### Store In A Cool Dry Place







# User Guide

How to use your tools

# How to use your **Janet Paww**

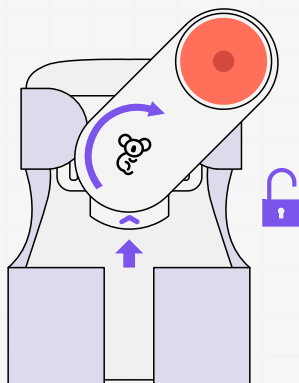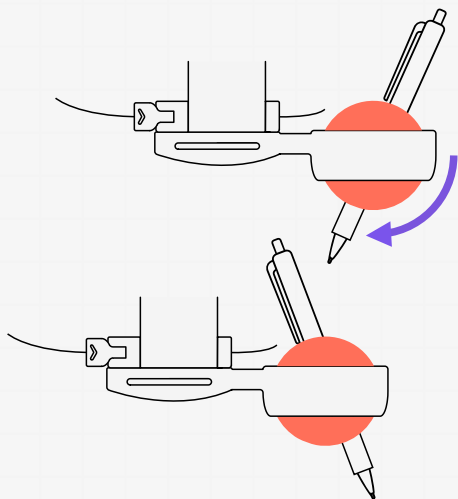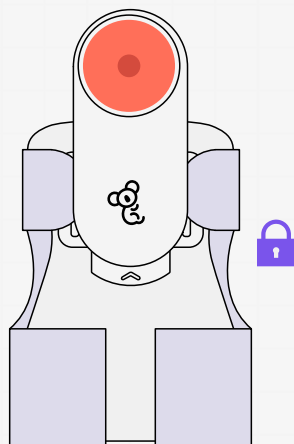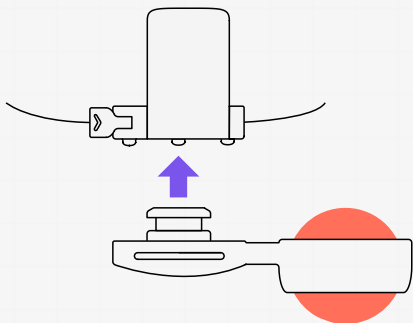





# How to use your **Amy Paww**

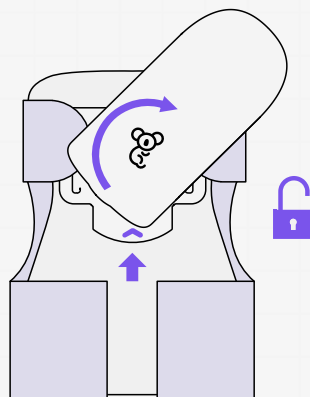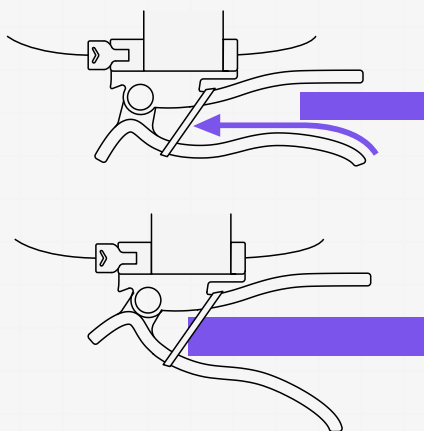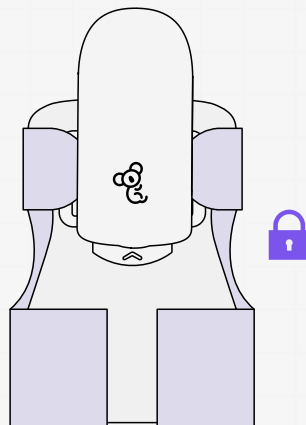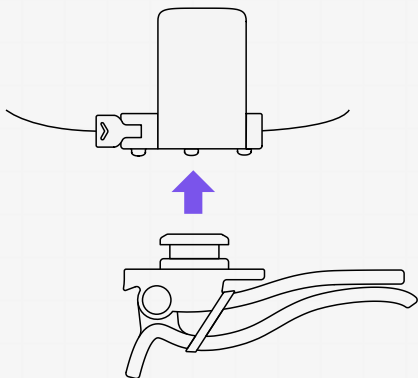



**0.5kg**






# How to use your **Grace Paww**

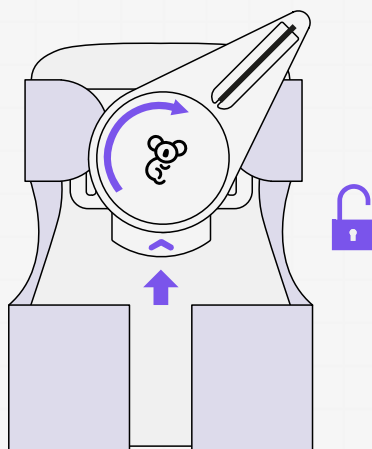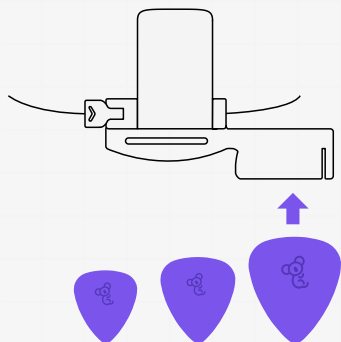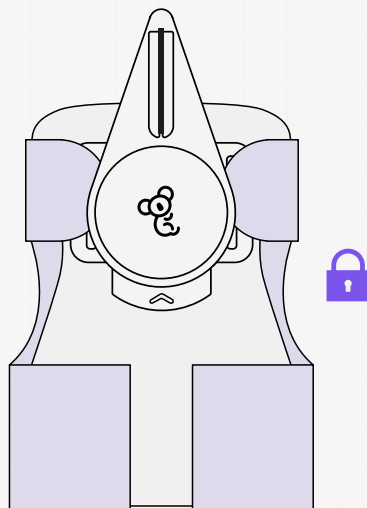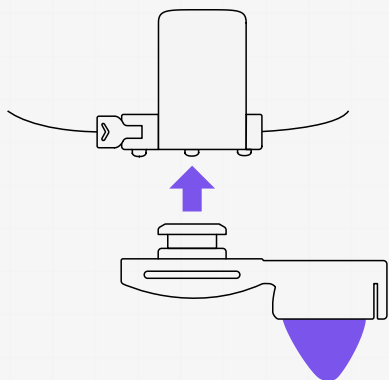

# How to use your **Rushton Paww**

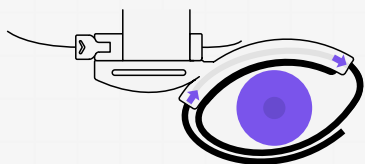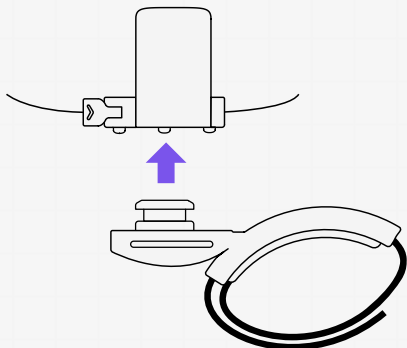

or




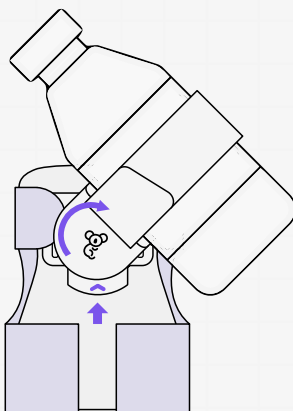
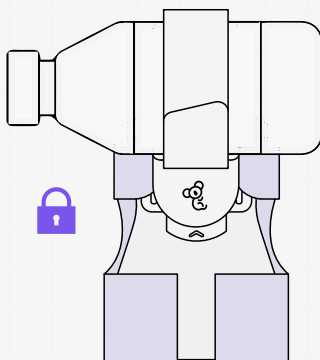
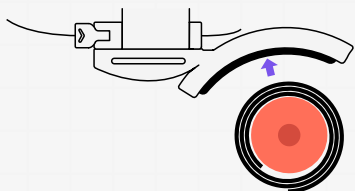


> 10kg





# How to use your **Sarah Paww**

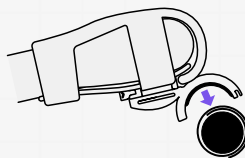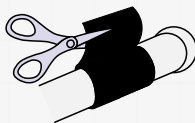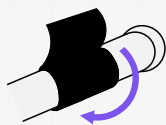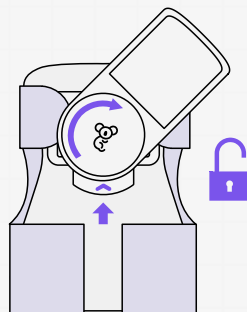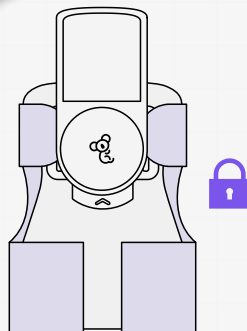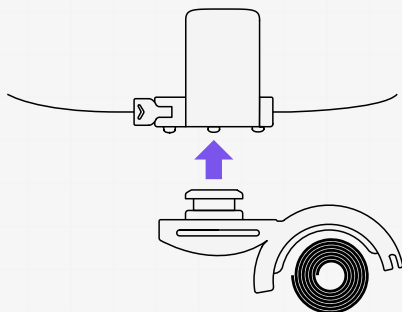

# Our Products

Find out about the ALX & Joeyy

# Meet The ALX

For Below Elbow

The ALX is the perfect fit for **below elbow and partial wrists**. Made up of a flexible sleeve with a soft fabric cover to hold it in place, the ALX is unlike any other prosthetic on the market. Whereas most rigid prosthetics rely on friction, suction or a pin lock mechanism, the ALX grips more like sportswear; strong, soft and snug.

The ALX sleeve is compatible with a wide range of detachable tools, opening the door to countless activities.



ALX Cover



ALX Socket

Sarah  
ALX Tool

Lockable  
Dock

# The ALX Tools



Janet  
ALX



Sarah  
ALX



Kitty  
ALX



Amy  
Paww



Rushton  
ALX



Grace  
ALX



Nicole Pro  
ALX



Nicole  
ALX



Joanie  
Tool



# Meet The Joeyy

For Under 3s

The Joeyy is a 'my first' prosthetic for **babies and toddlers from 0-3 years**. With its fun and friendly design, the Joeyy makes a superb practical and gentle introduction to the world of possibilities that prosthetics can offer.

The Joeyy, with no loose parts and a functional built-in tool, gives your child the freedom to explore the world around them, aiding grip on the handles of balance bikes and push-along walkers. But don't be fooled by their practical capabilities, the Joeyy also makes a fantastic puppet and is equally at home snuggled up with your little one on the sofa.

## Which design would you choose?



Dalmation



Turtle



Bumblebee



Sheep



Joeyy



# ALX & Paww Covers

The ALX and Paww sleeves are secured in place with a soft, fabric cover. Stand out or blend in, the choice is yours with our awesome range of covers designed in house by our very own Lauren and Esmé.




**Dinourmous  
Dinos**



**Rare  
Rainbow**



**Snazzy  
Superhero**

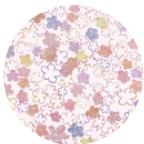

**Flower  
Power**



**Safari  
Adventure**



**Wildflower  
Meadow**



## Custom Covers

Wear a sleeve as unique as you are with Koalaa's bespoke cover design service.

Our design team will create a sleeve just right for you and you'll walk away with a completely unique cover for your Koalaa sleeve.

To get yours get in touch at  
**[hi@yourkoalaa.com](mailto:hi@yourkoalaa.com)**




# Cosmetic Hand & Covers

For times when you would like something more discreet, our cosmetic hands and covers can be matched to any skin tone. The realistic fingers on the hand can also be moved into different positions. Lastly, your long-sleeve clothing can be pulled down over your Koalaa prosthetic, leaving just in hand on show.



**Find the colour to  
match your skin**

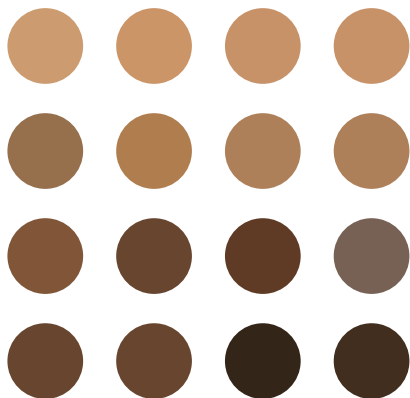



# Contact A Limb Buddy

Sometimes you want to ask a simple question, other times you just want to chat. At Koalaa there's no need for formal appointments and long waits, with our limb buddies available throughout the week you can get in touch any time.



**hi@yourkoalaa.com**



**+44 (0) 7482 165 098**



**@koalaacommunity**



## Want to return or replace your ALX?

If there are any issues with your ALX sleeve or tools contact your limb buddy directly or at **hi@yourkoalaa.com** and we'll do our best to solve any issues, replace the product or offer a full refund.



## Scan Me!

Scan the QR code to find our Koalaa Hub, here there is a digital version of all our Koalaa sleeve and user guides for you to use on the go!



# Join the Koalaa Community

Our users have become an incredible community who love to share their experiences with Koalaa and each other.

@koalaacommunity



## Send in your snaps!

We love to see how our users are getting on and what they're up to with their Koalaa sleeves. Send your stories, videos and photos to your limb buddy or email them to us at [hi@yourkoalaa.com](mailto:hi@yourkoalaa.com)



### Joanie

"I was really happy when Koalaa told me they were going to make a tool to help me throw the ball and that it was going to be called after me."



## Private Facebook Group

Join our private Facebook group and we'll accept your invitation as soon as possible. This is a closely monitored group so you can share all your photos with others safely.



The ALX has been awarded the following CE mark:  
**Medical Device Regulation MDR, EU 2017/745 Regulation**

This means it conforms with health, safety and environmental standards for products sold within the European Economics Area.