



# The ALX

## Fitting Guide



[www.yourkoalaa.com](http://www.yourkoalaa.com)


# What's In Your ALX Fitting Kitt?

The ALX is lightweight and breathable like clothing. It has a range of easily interchangeable tools designed specifically for the activities you love.



## The ALX Sleeve & Dock

Small, medium & large sleeve and docks

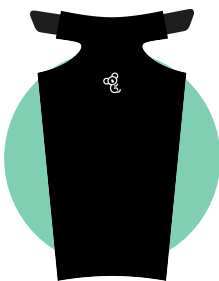

## ALX Cover

Small, medium & large ALX covers




## ALX Tools

ALX tools for a variety of activities



## Sports Straps

Velcro straps for securing limb in sleeve

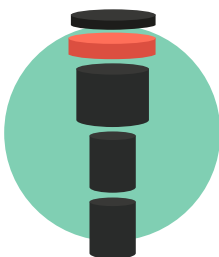

## Inner Foam

3 various density foams for inside the ALX sleeve



## Fitting Equipment


All the tools you need to fit an ALX

# The ALX Sizing Guide




**A**

Measure the circumference of the widest part of limb

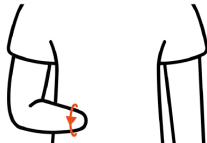


**B**

Measure the circumference of the narrowest part of limb

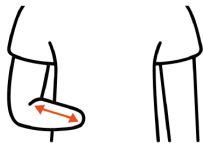

**C**

Measure length from crook of elbow to end of limb

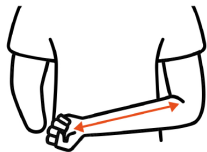

**N**

Measure length of opposite arm from crook of elbow to middle of palm

## Choosing a blank size

Use the following charts to determine the correct size of the blank based on the measurements taken.

Forearm Circumference **(Measurement A)**



**Blank Size**

14 - 19cm

**Small**

19 - 25cm

**Medium**


25 - 33cm

**Large**

33 - 41cm

**X-Large**

# Choosing a blank lenght

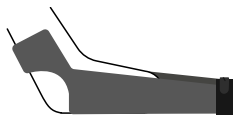

Or



Calculate for long sleeve

Use **Measurement N** - 1cm





Calculate for short sleeve

Use **Measurement C** + 7cm







# Choosing a dock size

Selecting a dock based on the length of the ALX blank cut to fit the user's limb. Note the blank tapers towards the end to resemble a forearm.







Blank Size	Small Dock	Medium Dock	Large Dock
Small	11 - 31cm		
Medium	25 - 37cm	17 - 25cm	11 - 17 cm
Large		29 - 37cm	11 - 29cm
X-Large			17 - 37cm

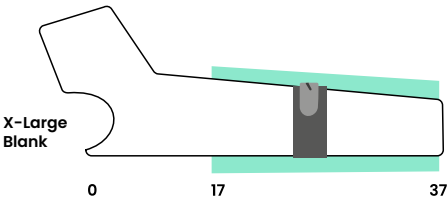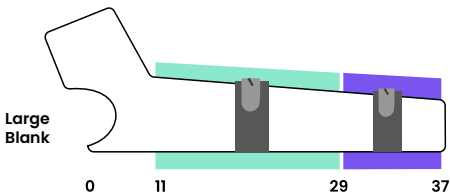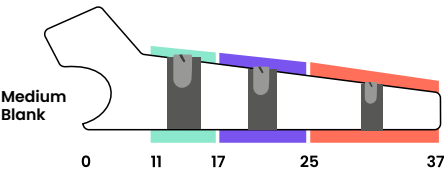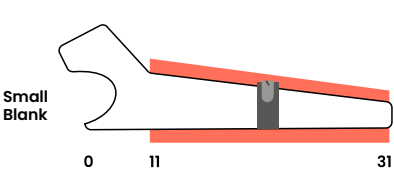

# Cutting a **full lenght** ALX

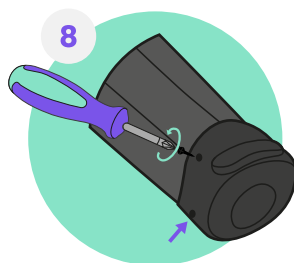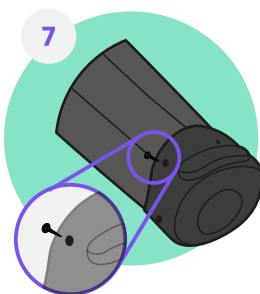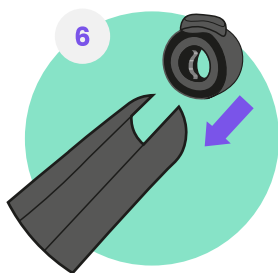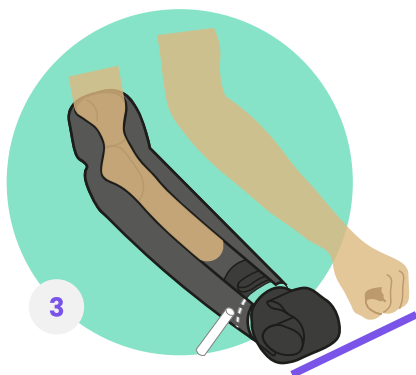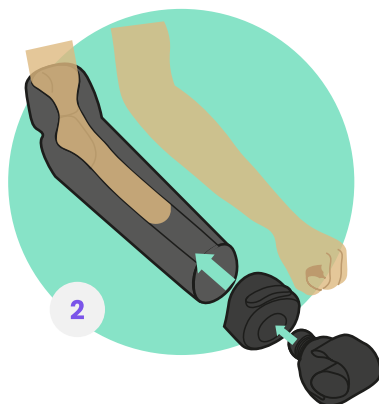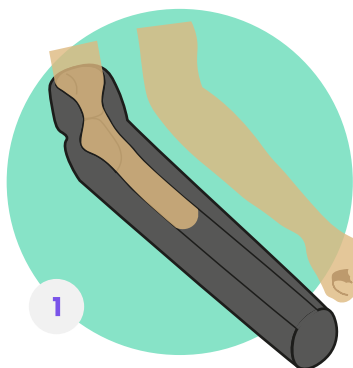

# Cutting a **short lenght** ALX

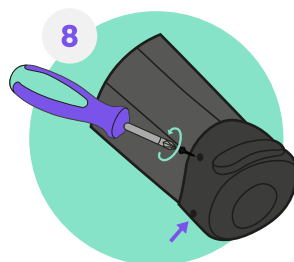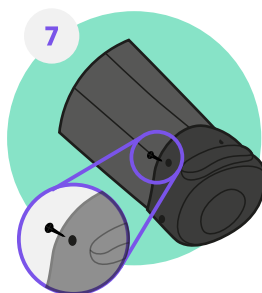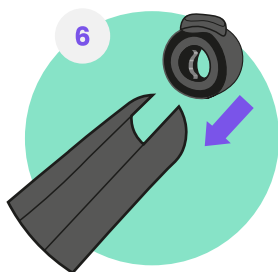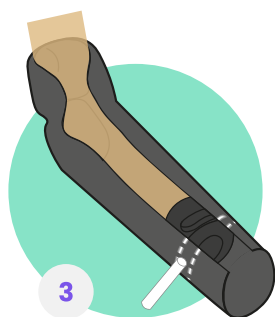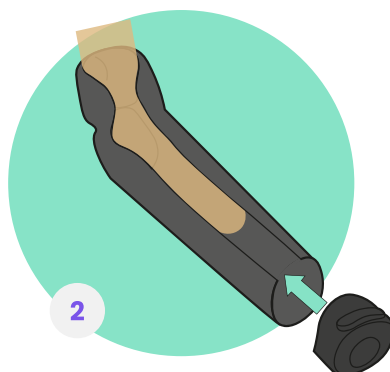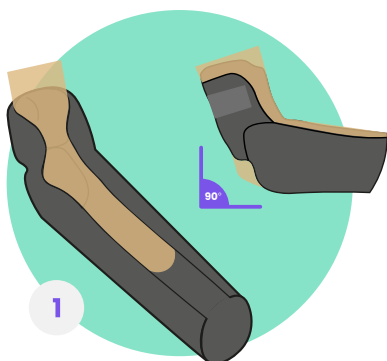


# Add foam padding




Soft Gel



Soft Foam

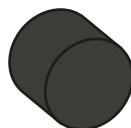

Firm Foam

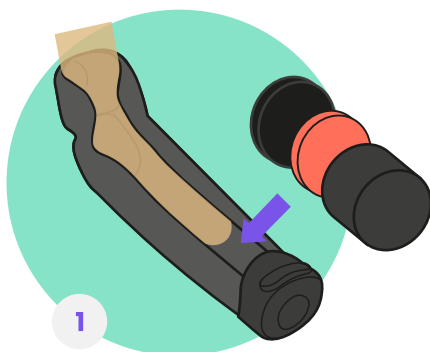

1




2

# Add support straps




1

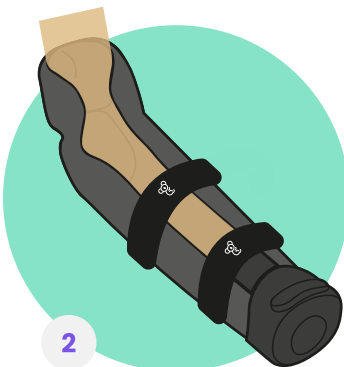

2

# Cut cover to length

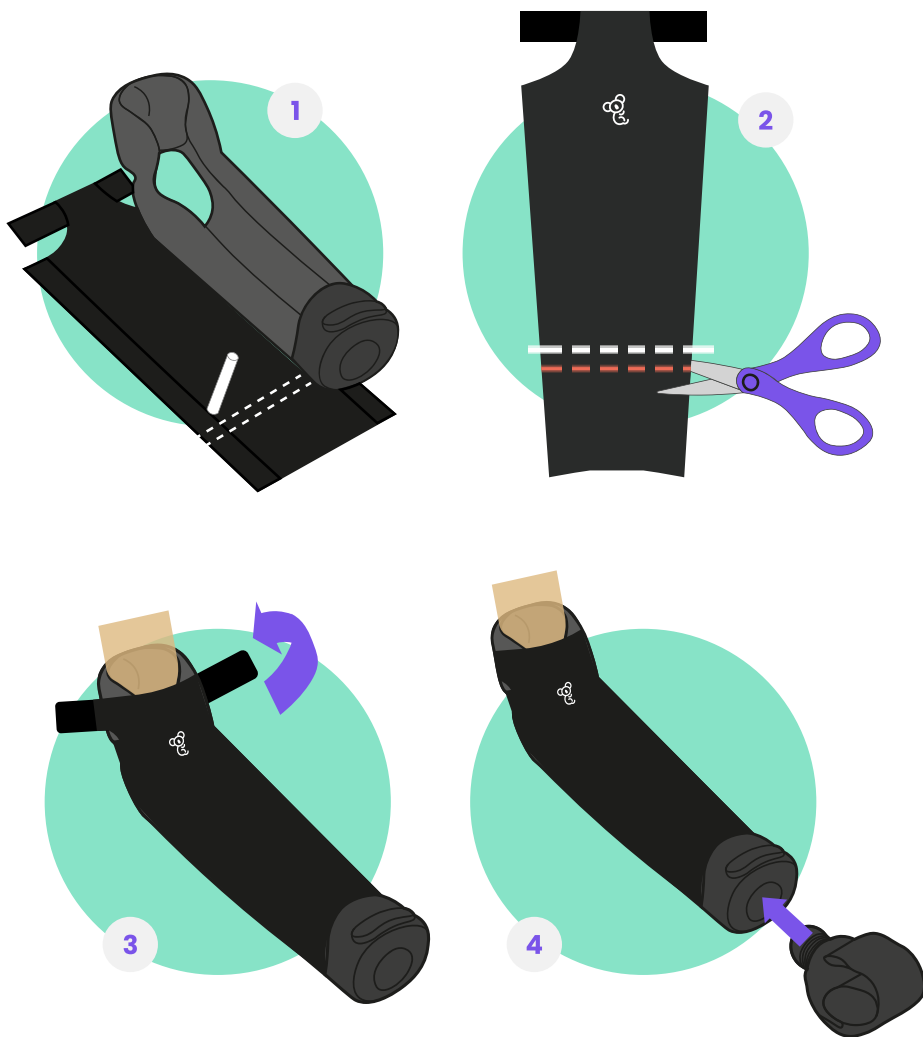



# Get In Touch

If there are any issues with your ALX sleeve or tools get in touch with a limb buddy directly or contact us at **hi@yourkoalaa.com** and we will do our best to solve any issues, replace the product or offer a full refund.



**hi@yourkoalaa.com**




**+44 (0) 7482 165 098**



**@koalaacommunity**

## Digital Guides

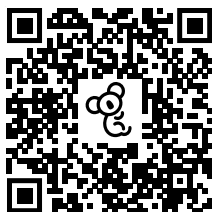

### Scan Me!

Scan the QR code to find our Koalaa Hub, here you'll find a digital version of all our Koalaa sleeve and user guides for you to use on the go!



The ALX has been awarded the following CE mark:  
**Medical Device Regulation MDR, EU 2017/745 Regulation**

This means it conforms with health, safety and environmental standards for products sold within the European Economics Area.